# Temporary cement

## IMPLANT RETAINED CROWNS

### Instructions for use

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DE

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## INSTRUCTIONS FOR USE

EN

### TEMPORARY CEMENT FOR IMPLANT RETAINED CROWNS

#### INDICATIONS

Specially formulated cement for luting of provisional crowns and bridges on implant abutments.

#### INSTRUCTIONS FOR USE

- 1) Finish and polish the restoration. Check occlusion and proximal contacts and complete final adjustments.
- 2) Dry and isolate the area being treated. Absolute dryness of the restoration is not necessary, but contamination should be avoided.
- 3) Make sure that the restoration is clean and dry. It is advisable to apply a thin layer of water-based lubricant to the abutment surface to facilitate future removal of implant restorations.
- 4) Bleed the syringe before attaching the automix tip. Bleed the syringe again after placing Proclinc Expert Temporary Cement's automix tip to ensure correct flow of base and catalyst components. Place a thin layer of cement in the restoration. Remember that the working time for this cement is between 45-60 seconds.
- 5) Keep the crown in the desired position for 2-2.5 minutes by asking the patient to close on a cotton ball to ensure accurate, secure placement. Excess cement can safely be removed after 2.5 minutes.
- 6) Total setting time of the cement is between 4 - 5 minutes, depending on conditions.

#### PRECAUTIONS

Though uncommon, some individuals may be sensitive to this product. If an allergic reaction occurs, discontinue use, remove from the patient's mouth and refer patient to a doctor.

Important application information for Proclinc Expert Temporary Cement:

only a small bead of cement around the margin is necessary to securely cement highly retentive, provisional restorations. Using less cement saves money and allows easier removal of the temporary restoration.

#### COMPOSITION

- Multifunctional methacrylates
- Urethane dimethacrylate
- Catalysts

#### STORAGE

- Keep refrigerated
- For dental use only

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