<b><i>Instructions For Use:</i></b>	<b>en</b>
<p><b>Contraindications:</b></p> <ul style="list-style-type: none"><li>Do not use TotalFill Bioceramic Root Repair Material in patients with a known allergy to any of the product’s ingredients. An allergic reaction may require re-treatment.</li></ul>	
<p><b>Precautions:</b></p> <ul style="list-style-type: none"><li> Do not use after the expiration date</li> <li> Consult accompanying Safety Data Sheet (SDS)</li> <li> Do not re-use the disposable syringe tips (BC Tips)</li> <li>Discard the BC Tips after each application. Potential cross contamination may occur if single use syringe tips are reused or not properly cleaned before use.</li> <li>TotalFill Bioceramic Root Repair Material and BC Tips are not designed to be sterilized. Failure to follow these instructions could distort the tips and/or damage the product resulting in procedural delays or user inconvenience.</li> <li><b>Cleaning:</b> <ol style="list-style-type: none"> <li>Disinfect the exterior surfaces of the syringe and syringe cap (once it is tightly sealed onto the syringe) prior to storage to reduce the risk of cross-contamination.</li> <li>The TotalFill Bioceramic Root Repair Material syringe should be mantled with a hygienic single-use barrier sleeve for infection control for direct intra-oral use.</li></ol></li> <li>Ensure there is continuous flow of material and the placement site is completely filled. Failure to do so may result in procedural delays.</li> <li>Ensure that any bleeding is under control prior to placing the Total-Fill Bioceramic Root Repair Material as the material may wash out of the placement site and require re-treatment.</li> <li><b>Storage:</b> <ul style="list-style-type: none"><li> Keep Dry</li> <li> Store at room temperature</li></ul></li> <li>Closely follow the recommended storage conditions. Failure to do so will cause the material to prematurely set resulting in re-treatment of material placement or user inconvenience. To avoid prematurely inducing the setting process closely follow these guidelines: <ol style="list-style-type: none"> <li>Use the cap to keep the jar or syringe tightly closed when the material is not in use. Keep the cap free of moisture.</li> <li>Keep TotalFill Bioceramic Root Repair Material tightly sealed in its pouch and store at room temperature in a dry area to avoid moisture contact.</li></ol></li></ul>	
<p><b>Warnings:</b></p> <ul style="list-style-type: none"><li> Irritant</li> <li> Do not use if package is damaged</li></ul>	
<ul style="list-style-type: none"><li>Use personal protective equipment to avoid contact of TotalFill Bioceramic Root Repair Material with the skin, mucus membranes and eyes. Unset material may cause irritation.</li> <li>Do not use excessive force to apply the material into the root canal as this may cause patient sensitivity/discomfort or breakage of the syringe plunger.</li> <li>Always inspect the syringe prior to application into the site. Using a syringe with illegible reference markings could lead to overfilling or underfilling of the root canal site.</li> <li>TotalFill Bioceramic Root Repair Material has not been tested on pregnant or nursing mothers.</li> <li>Procedural delays or user inconvenience may be experienced if the BC Tip is not inspected prior to use. If the material does not flow out of the syringe tip or if the syringe tip feels stiff, discard the tip and use a new one.</li> <li>Always check the expiration date of the product to prevent procedural delays or user inconvenience (e.g. material becomes brittle or will not set).</li> <li>Overfilling the root canal may lead to patient sensitivity, foreign body inflammation, maxillary sinus aspergillosis, paresthesia of an-esthesia due to nerve impingement or may require surgical removal of the overfill.</li> <li>Carefully read package labeling to ensure use of the appropriate bioceramic material. Failure to do so may cause user or patient inconvenience.</li> <li>Multiple continuous applications of material using the syringe delivery system may cause hand fatigue.</li></ul>	
<p><b>Product Description:</b></p> TotalFill Bioceramic Root Repair Material is a ready-to-use, premixed bioceramic material developed for permanent root canal repair and surgical applications. It is an insoluble, radiopaque and aluminum-free material based on a calcium silicate composition, which requires the presence of water to set and harden. TotalFill Bioceramic Root Repair Material does not shrink during setting and demonstrates excellent physical properties.	
<p>TotalFill Bioceramic Root Repair Material is packaged in a preloaded syringe (BC RRM and BC RRM Fast Set Putty) or in a jar (BC RRM Putty).</p>	
<p><b>Indications For Use:</b></p> <ul style="list-style-type: none"><li>Repair of Root Perforation</li> <li>Repair of Root Resorption</li> <li>Root End Filling</li> <li>Apexification</li> <li>Pulp Capping</li></ul>	
<p><b>Working Time:</b></p> No mixing is required. The setting reaction begins as soon as the material is placed in contact with a moist environment. The working time is more than 30 minutes.	
<p><b>Setting Time:</b>  <b>BC RRM and BC RRM Putty</b>  Setting time is a minimum of 2 hours in normal conditions, but can take longer to set in extremely dry canals.</p>	
<p><b>BC RRM Fast Set Putty</b>  Setting time is approximately 20 minutes. In extremely dry conditions the setting time may be longer.</p>	

## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

# TotalFill®

## Bioceramic Root Repair Material™

## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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# TotalFill®

## Bioceramic Root Repair Material™

## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

# TotalFill®

## Bioceramic Root Repair Material™

## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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## Instructions For Use

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## Brugsanvisning

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