Sterilisation pouches

- de Gebrauchsanweisung
- en Instructios for use
- es Instrucciones de uso
- fr Mode d'emploi
- it Istruzioni per l'uso
- pt Instruções de uso
- nl Gebruiksaanwijzing
- pl Instrukcja postępowania
- da Brugsvejledning
- ro Instrucțiuni de utilizare
- sv Bruksanvisning
- no Bruksanvisning
- cs Návod k použití
- sk Návod na použitie
- sl Navodila za uporabo
- et Kasutusjuhised
- hu Használati utasítás
- hr Upute za uporabu
- bg Инструкции за употреба
- el Οδηγίες χρήσης
- fi Käyttöohjeet

proclinic
EXPERT

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en Instructios for use / Sterilisation pouches

Self-seal, non-sterile pouches are suitable for packaging items that need to undergo a sterilisation process.

- Check the size of the item to be sterilised.
- Insert the item through the unsealed side of the pouch.
- Remove the protective liner from the adhesive strip along the pre-folded line. Press from the centre outwards to ensure the pouch is fully sealed.

The indicator will change colour to brown after sterilisation, helping you distinguish processed packs from those not yet processed. Please note that a colour change does not guarantee that the contents have been properly sterilised; it is the user's responsibility to verify successful sterilisation.

Avoid direct exposure to sunlight or heat sources, and do not store in heavily polluted environments. Store the packaging off the floor and away from walls, as the material is sensitive to humidity and heat. If the outer packaging has been opened, restore the original packaging conditions.

The product's characteristics and properties will remain unchanged provided that appropriate conditions for storage, use and preservation are maintained. Do not use if the packaging is damaged or has been stored incorrectly. Each pouch is single-use and must not be reused. Suitable sterilisation methods include steam and ethylene oxide. Handle with care.

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