FICHA TÉCNICA

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TECHNICAL DATA SHEET

BIOLOGICAL INCUBATOR D-INCUBATOR TEST PLUS BRAND: D-DEVICES		
REFERENCES	91005	
MANUFACTURER	TECNO - GAZ S.p.A. Str. Cavalli n ° 4 - Baganza Room - Parma - ITALY. Tel. +39 0521 8380 - Fax +39 0521 833391 e-mail: info@tecnogaz.com http://www.tecnogaz.com	
DESCRIPTION	The biological incubator is a product developed for the incubation of biological indicators.	
COMPOSITION	The incubator consists of a frame, an ABS cover, an aluminum heating unit with 3 compartments for the test tubes / biological indicators and an electronic card for automatic temperature management of the heating unit. On the side there is the connection cable with the autoclave and the start button, while on the front of the equipment there is the warning light.	
PACKAGING	1 unit per container.	
INTENDED USE	The biological incubator is a product developed for the incubation of biological indicators at 57 ° C.	
INSTRUCTIONS FOR USE	See instructions on the product.	
HANDLING AND STORAGE CONDITIONS	The equipment does not require special maintenance, except for periodic cleaning of the heating unit to prevent substances from depositing in the compartments of the biological indicators. • If you have just used the machine, wait for it to cool down. • Use a sponge slightly dampened with soap and water, drying with a clean, dry cloth. • Do not use harsh cleaners that could irreversibly alter the appearance of the plastic parts of the machine. • Do not use foggers to directly spray the machine, as this could damage the electrical system inside.	

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