FICHA TÉCNICA

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TECHNICAL DATA SHEET

ORTO PROCLINIC RESIN POWDER 1KG PROCLINIC		
REFERENCES	H53911	
MANUFACTURER	retec® Kunststofftechnik GmbH Industriestraße 2 D-61191 Rosbach v.d.Höhe (Rodheim)	
CLASSIFICATION	Class IIa Medical Device, under Directive 93/42/EEC.	
DESCRIPTION	It is a self-curing orthodontic resin based on methyl methacrylate, indicated for the manufacture of orthodontic appliances and splints by spraying or spreading.	
COMPOSITION	The resin comes in powder and liquid form, with the powder being the main component at 1 kg. It is self-curing, which means that it hardens without the need for additional external heat. The liquid monomer required for mixing is methyl methacrylate-based and is typically combined with the powder in a standard ratio of 10 g of powder to 7 g (≈7.4 ml) of liquid. The mixture allows for easy processing, great fit and long-lasting results. <u>Advantages:</u> Quick and easy to use for making orthodontic appliances and splints. Long-lasting results with a great fit. Compatible with methyl methacrylate monomers and spray or moulding techniques. Cadmium-free and biocompatible when used correctly.	
PACKAGE	1 unit of 1 kilo.	
INTENDED USE	The manufacture of orthodontic appliances, including brackets, orthodontic plaster and other devices using the spreading or spraying method. The manufacture of bite splints, anti-snoring devices or other customised types.	
STORAGE AND TRANSPORT	Store in a cool and dry place. Keep away from sunlight. Protect from moisture and avoid excessive heat.	